FLOCARE® INFINITY™ III INSTRUCTIONS FOR USE

RATE THERAPY



First check you have the correct tube feed and that it's in date



Attach Flocare Infinity
Pack giving set



Screw on tightly; hang the bottle or container on the back of the stand using the z-hook or place in go-frame



Open pump door and insert giving set



Close door












Set rate by pressing the +/- buttons. Press the middle button to confirm

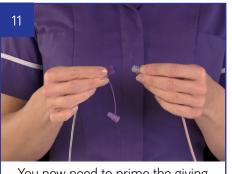


You now need to prime the giving set; make sure the giving set is not connected to the tube



Press and hold the fill set button, after 2 seconds the pump will start


The fill set button should only be used to fill a new and empty air-filled giving set.

Do not use the fill set function for any other purpose.

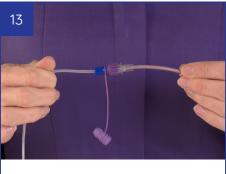

Connect the giving set to the feeding tube



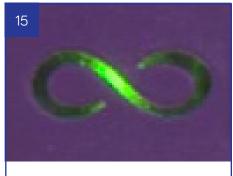

The light on the run symbol indicates the pump is working



Press info button for additional information



Press alarm button to silence alarm. For further information on alarms please see troubleshooting guide

For more information call 0800 093 3672 or for online training visit NUTRICIAFLOCARE.COM

For more information call 0800 093 3672 or for online training, visit nutriciaflocare.com

Information contained here relates to programming the pump only. For clinical queries contact your healthcare professional. This information is intended for patients or carers of those who have been prescribed a tube feed by a healthcare professional.