

# NUTRICIA FLO CARE® INFINITY™ PUMP

## CLEANING GUIDELINES

### IMPORTANT — PLEASE AVOID:

Immersing pumps in water, detergents or other fluids.

Storing the pump in extreme heat or cold (under -40°C up to +50°C).



Pumps may be wiped with a wet cloth and detergent and rinsed under running water.



### CLEANING THE PUMP:

Clean and remove any tube feed residue from all sides, the sensor area, and in the corners of the keypad where it meets the pump housing. Use a mild, diluted cleaning solution and damp cloth. Avoid immersing pump. If necessary, use a moist cotton bud to clean sensor area.

### DRYING THE PUMP:

Shake pump to remove excess water and allow to air dry for approximately 30 minutes or until completely dry, before plugging in to charge.



This information is intended for patients or carers of those who have been prescribed a tube feed by a healthcare professional.