



# GO-FRAME INSTRUCTIONS FOR USE






## SETTING UP THE GO-FRAME

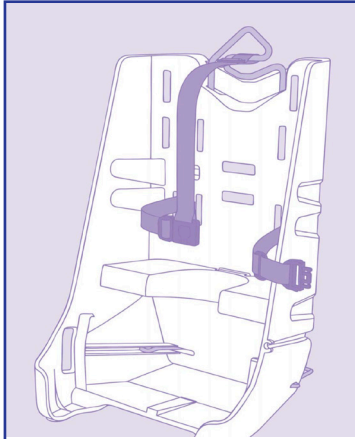





Although the shelf is moveable, all bottle sizes fit the lowest position.

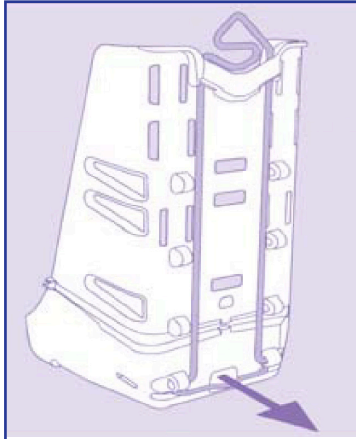

When using without a go-bag (back-pack), pull out the stand at the bottom of the go-frame, until it clicks into position, to make it more stable.

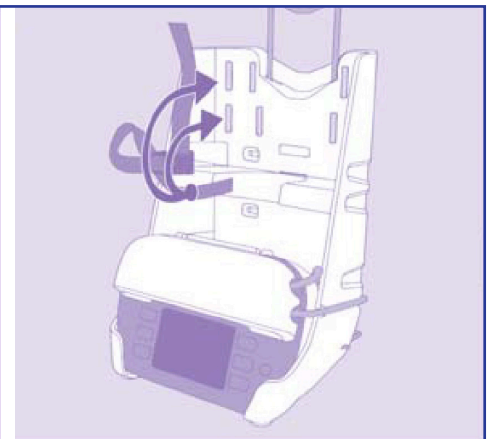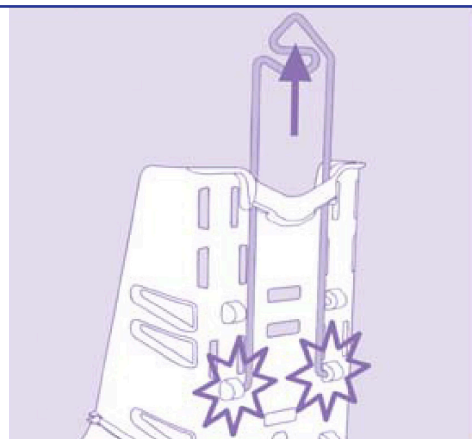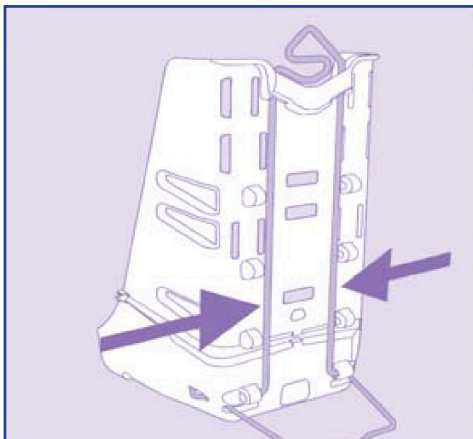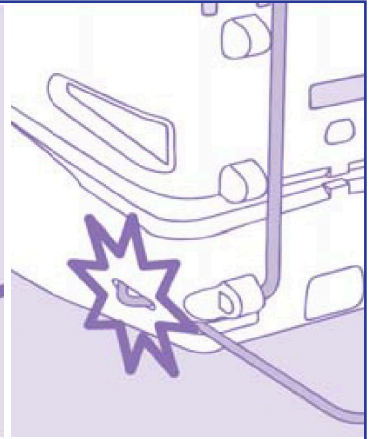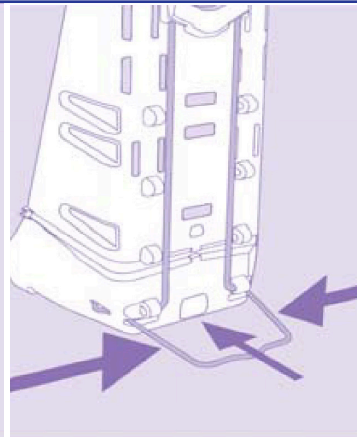

The stand at the back can be moved up for 1000ml OpTri bottles (ensure it clicks into place), remember to also move the straps up to the same level. For all other bottle sizes, it should remain at the 'default' lowest position.

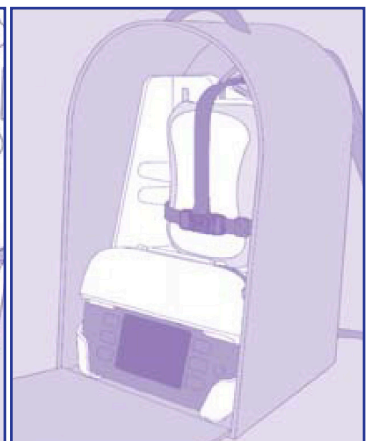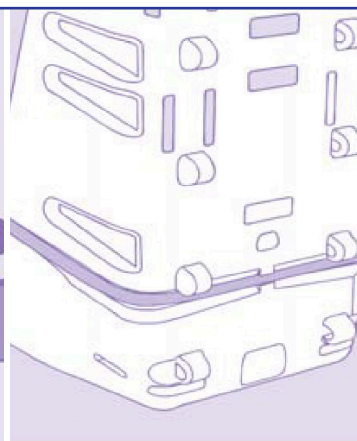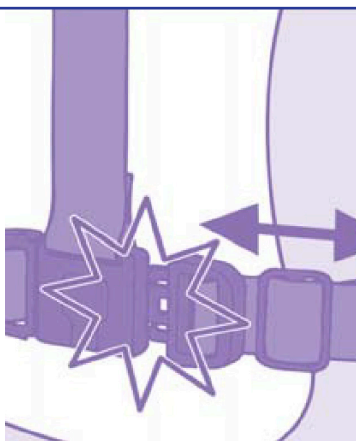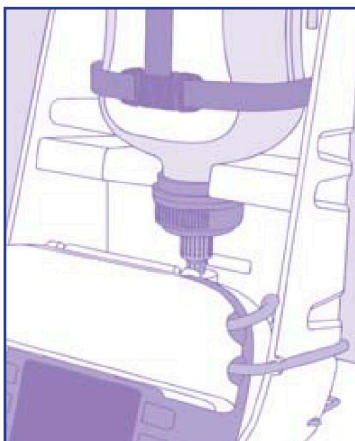

Connect the bottle and pump to the giving set; position the bottle onto shelf and sit the pump below, at the bottom of the go-frame. Using the straps provided, click the bottle into position, then wrap the giving set into position.

If using with a go-bag, simply push the stand back in and place into go-bag.