


NUTRICIA

flocaire[®] Infinity[™] III

HOW TO USE YOUR NEW FEEDING PUMP

Your new pump is there to make things a bit easier for you. Remember, using your pump shouldn't mean you have to miss out on anything you like to do. Even if you are not allowed to eat normal food, you can still have **lots** of fun!

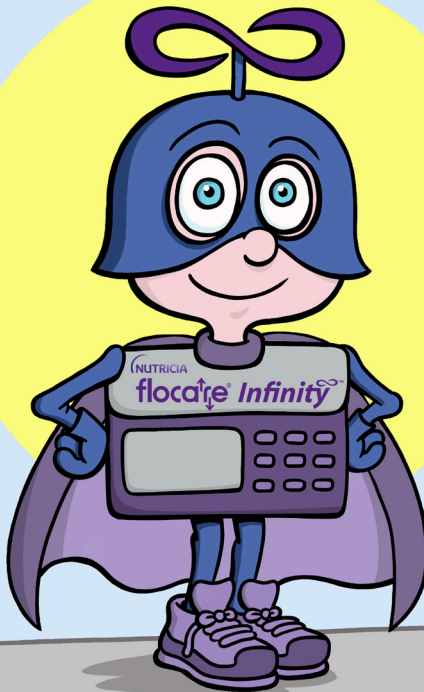

WHAT WORDS MEAN

Before you start using your pump, learn about some strange new words

Volume, Flow rate, Speed	Tells you how fast your pump is running
Volume, Dose	Tells you how much feed is given
Giving set	This is the tubing that you connect to your pump and tube feed. It goes through the pump and connects to your feeding tube
Continuous feeding	A feed given over a long time, like lots of meals joined into one
Bolus feeding	A feed given over a short time, like one meal



YOUR PUMP IS
SMALL, YOU CAN
TAKE IT WITH YOU
WHEREVER YOU GO



IF YOU DON'T HAVE A
BACKPACK CONTACT
YOUR NUTRICIA
HOMEWARD NURSE

HOW TO SET UP YOUR PUMP

Your new pump is quite easy to use — these steps will teach you how to use your pump. Please ask an adult if you need help at any time.

STEP 1

Open the feed bottle; take the giving set and screw it tightly onto the bottle, so the spike goes through the silver foil; hang the feed up



STEP 2

Open the pump door by pressing on the door clip and lift the door at the same time. Put the loop part of the giving set around the big wheel, pulling slightly on the giving set to clip the plastic part into place; close the pump door



STEP 3



Press the **ON/OFF** button and hold it down, count slowly to three; the pump is on when you hear a beep



Make sure you are in the home screen, if not press the **HOME** button



Press the **THERAPY** button
You are now ready to set up your personal feed programme



Select **VOLUME-RATE** from the list

HOW TO SET UP A VOLUME-RATE FEED

- Select **volume*** and set volume by pressing +/- buttons, press the middle button to confirm
- To set the rate (or speed) your pump will run at, select **rate*** and set rate by pressing +/- buttons, press the middle button to confirm

**This is the volume and rate your dietitian or nurse has asked you to use*



Press the **FILL SET** button and hold it down, count slowly to three; the pump will start and the whole tube will fill up with the feed. You can stop the tube filling up with feed by pushing the FILL SET button again



Press **START** button to start feeding

IF YOU NEED TO STOP YOUR PUMP, press the **START/STOP** button once; the pump will stop when you hear three beeps

TO SWITCH YOUR PUMP OFF when the feed has finished, press the **ON/OFF** button and hold it down, counting slowly to three until you hear a beep — the pump is now off

Read through the steps again when you need to and don't forget that your dietitian or nurse will be happy to answer any questions you might have.
Happy feeding, and don't let your pump stop you from having fun!

NUTRICIA
flocafe[®] Infinity[™]

Nutricia Ltd
White Horse Business Park, Trowbridge, Wiltshire BA14 0XQ

SCC3914-11/20

