

FLOCARE BOLUS SET

INSTRUCTIONS FOR USE & CLEANING INSTRUCTIONS

You will need:


- Flocare Bolus Set
- Feed as prescribed
- Water for flushing
- A 60ml syringe

Check all expiry dates and wash hands thoroughly


Open the packaging and lay contents on a clean dry surface



1 Attach a 60ml syringe to the bolus set

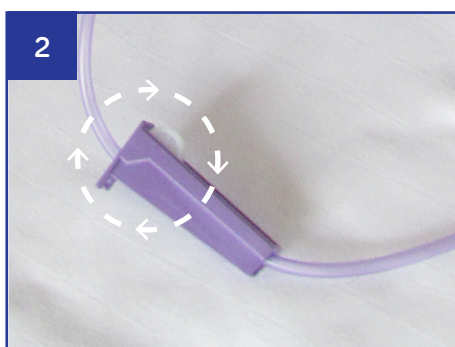


2 Open the roller clamp and flush with water as directed to expel any air

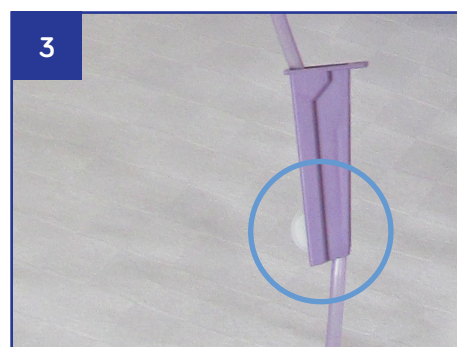


3 Close the roller clamp



4 Remove the syringe from the bolus set

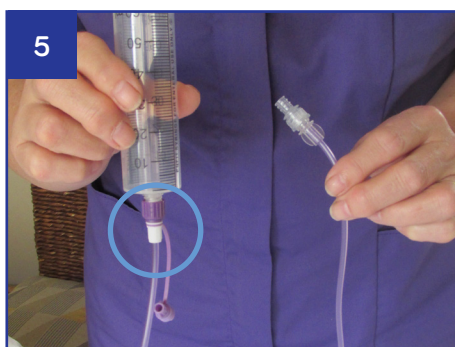

5 Attach the barrel of the bolus set to the extension set

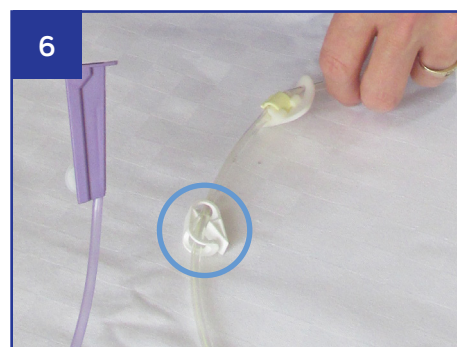


6 Close the clamp on the feeding tube (if appropriate)



7 Attach the extension set to the feeding tube



8 Pour the prescribed feed into the barrel and open the roller clamp

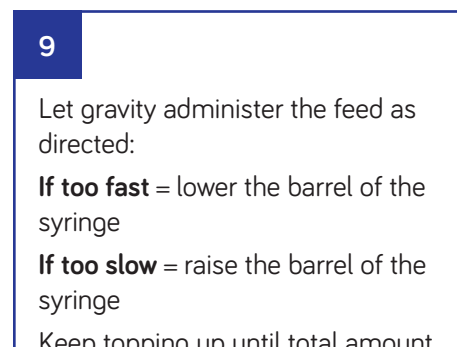

9 Let gravity administer the feed as directed:




If too fast = lower the barrel of the syringe

If too slow = raise the barrel of the syringe

Keep topping up until total amount of prescribed feed has been administered

FLOCARE BOLUS SET

INSTRUCTIONS FOR USE & CLEANING INSTRUCTIONS

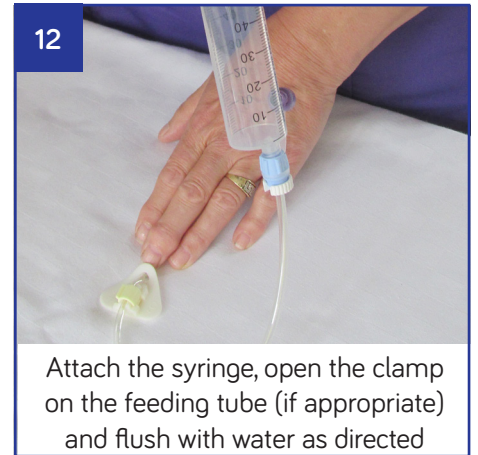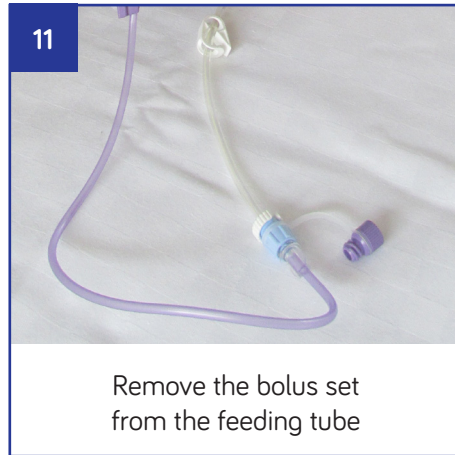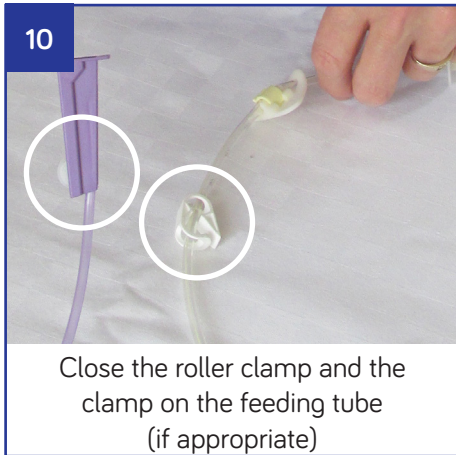

Discard bolus set as advised by your healthcare professional or Nutricia Homeward Nurse Specialist

The following cleaning instructions must be followed immediately after use of a bolus set.

❶ Do not allow traces of feed to dry in the bolus set as they might not be removed by cleaning

❷ Use warm soapy water to wash the syringe and the tube of the bolus set

❸ Rinse thoroughly under cold running tap water to remove all traces of the soapy water

❹ Allow to air dry

❺ Once completely dry store in a clean dry place

❻ The bolus set may be cleaned and re-used for no longer than 24 hours