

# FLOCARE<sup>®</sup> 2-PACK CONNECTOR

## INSTRUCTIONS FOR USE

### YOU WILL NEED:


- Flocare 2-Pack Connector
- Flocare giving set
- Tube feed, as prescribed

Check all expiry dates and wash hands thoroughly

Open the packaging and lay contents on a clean dry surface




1

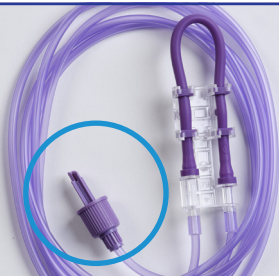

Remove the dust cap from the Flocare giving set and discard

2

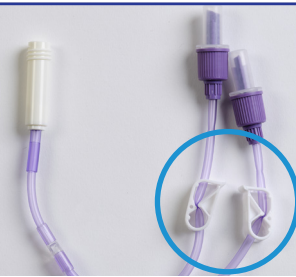

Close both the clamps on the Flocare 2-pack connector

3



Attach the white port of the Flocare 2-Pack connector to the end of the giving set and screw together tightly

4



Remove the dust caps from both ends of the 2-Pack connection

5



Remove the cap from both bottles of the prescribed tube feed (or water)

6



Attach the 2-Pack connector port to each of the bottles of prescribed tube feed (or water)

7



Using two z-hooks, hang the two bottles of feed on the stand by placing them back-to-back

8



Insert the giving set into the pump and open the clamps on the 2-Pack connector

Programme the pump as advised by your healthcare professional or Nutricia Homeward Nurse

\*Please note, you will need to use the 'fill set' facility to prime your feed to the end of the giving set before you attach it to your feeding tube