

# HOW TO USE THE UNIVERSAL ADAPTOR FOR 24 HOURS WHEN USED WITH STERIFEED REUSABLE BOTTLES

Wash hands thoroughly, mix or decant feed as usual, then connect the Flocare Infinity giving set and universal adaptor to the Sterifeed bottle.



1  
If the next feed is not due within the next 30 minutes, disconnect the used giving set and adaptor from the empty bottle and connect it to a new clean bottle before storing the empty reusable bottle, adaptor and giving set in the fridge.



2  
Rinse and wash your used bottle as soon as possible.



3  
When it's time for your next feed, remove the reusable bottle and giving set from the fridge and decant/mix the new feed as usual. Now connect the adaptor and giving set you used previously. You can also use a new giving set if you have one available.



4  
Every time you connect and disconnect, make sure to replace or remove the small cap and keep it safe.



5  
The empty clean reusable bottle and attached giving set should be stored in the fridge in between each use.



6  
Make sure you discard the adaptor and giving set after 24 hours.




7  
Wash and sterilise your reusable bottles as instructed by your managing healthcare professional. The reusable bottles should be used for up to 30 times and please remember to recycle these when reaching this point.



# HOW TO USE THE UNIVERSAL ADAPTOR FOR 24 HOURS WHEN USED WITH A PREPARED FEED

Examples of these are, but not exclusive to:

Note: Not all 125ml and 200ml bottles are compatible with the adaptor.

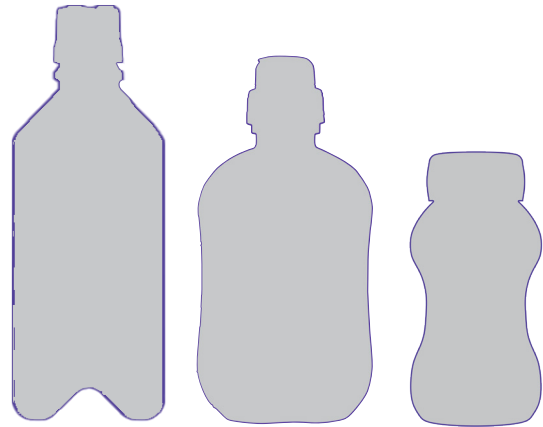

1 Wash hands thoroughly, check feed and expiry date.



2 Attach the Flocare Infinity giving set and universal adaptor to your prescribed feed or water as usual.



3 The prepared feed bottle (with or without feed remaining) and attached adaptor and giving set can remain in the frame or stand in between use.



4 In between each feed, replace the cap on the end of the giving set to protect it from getting dirty. You can replace your giving set with a new one if you have one available.



5 Make sure you discard the adaptor and giving set after 24 hours.



6 Remember to recycle where possible.

**If you would like more information, or have any questions, please contact the Patient and Carer Helpline on 03457 623653.**

This information is intended for patients, and carers, who have been prescribed a tube feed by a healthcare professional.

All products shown are Foods for Special Medical Purposes for the dietary management of disease related malnutrition and must be used under medical supervision.

Nutricia Limited  
Newmarket Avenue, White Horse Business Park, Trowbridge, Wiltshire BA14 0XQ

AMN0716-01/23

