

HOW DO I MEASURE THE PH OF GASTRIC ASPIRATE


IMPORTANT NOTE: It is important that you contact your Nutricia Homeward Nurse if you need additional training on the Flocare pump (if applicable) or tube feeding.

This leaflet provides specific guidance on how to measure the pH value of gastric aspirate when confirming the position of your child's feeding tube.

Be aware that pH testing of gastric aspirate should not be used by itself to determine your tube is in the right place. You should always consider other factors, such as the external tube length and how your child is feeling in themselves, e.g. any pain, redness or swelling around the tube site, any unusual abdominal pain, chest pain, breathing difficulties, or fever.

If you have any reason to suspect your child's feeding tube is positioned incorrectly, do not use the tube and contact a healthcare professional immediately.

NOTE: This guidance relates to gastrostomy tubes (e.g. PEG, BGT, RIG, LPD/Button) only.

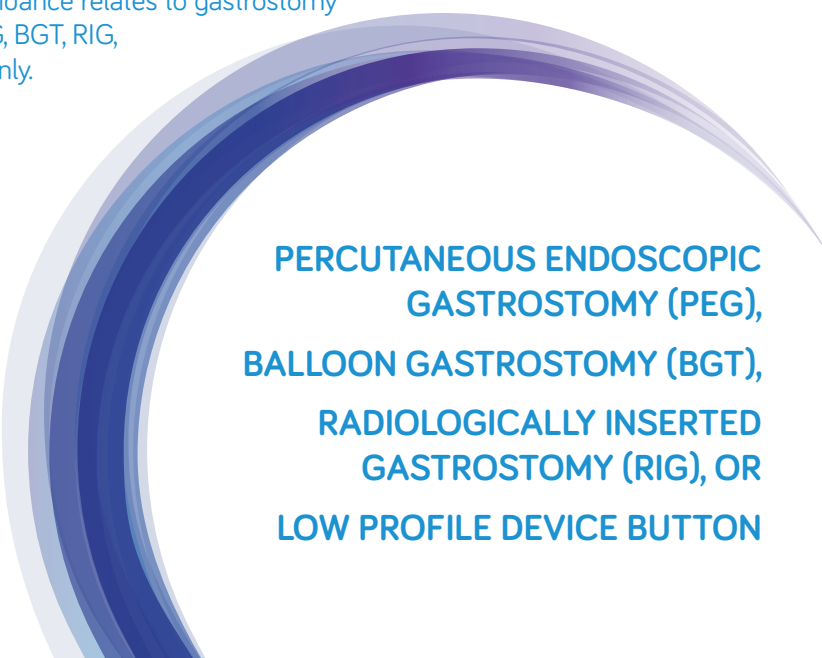

**PERCUTANEOUS ENDOSCOPIC
GASTROSTOMY (PEG),
BALLOON GASTROSTOMY (BGT),
RADIOLOGICALLY INSERTED
GASTROSTOMY (RIG), OR
LOW PROFILE DEVICE BUTTON**

HOW DO I MEASURE THE PH OF GASTRIC ASPIRATE?

Where enteral nutrition/medication has already passed through the tube, wait at least one hour without further feeding before pH testing the gastric aspirate.

Equipment:

- 60ml enteral syringe
- pH indicator strips (CE marked for human gastric aspirate and in date)
- Water (type and volume as recommended by the managing healthcare professional)
- Extension set (if using a Button).

Process

- Wash hands before and after handling your child's feeding tube.
- Confirm your child's feeding tube remains in the correct position.
- Attach extension set (if feeding tube is a Button).
- Ensure the clamp (if present) on the tube is closed before removing end cap. Attach the 60ml enteral syringe to the end of the tube
- Open the clamp on the tube (if present) and very slowly pull back on the syringe plunger until a small amount of fluid (at least 0.5–1.0ml) appears in the syringe
- Detach the syringe from the tube, replace the end cap of the tube and open the clamp. Detach extension set (if feeding tube is a Button)
- Drop the fluid onto the pH indicator strip and measure the pH value as per manufacturers instructions.

If the pH value is 1-5.5 it suggests the tube is in the correct position (the stomach). However, it is important to consider all other relevant factors as described during training with the Nutricia Homeward Nurse. Once you are confident the tube is in the correct position flush the tube with water (type and volume as recommended by the managing healthcare professional).

If the pH value is more than 5.5, do not administer anything via the tube. Check the pH value again in 30-60 minutes. If the pH value remains above 5.5 contact the managing healthcare professional. It is very important that you do not administer anything including enteral nutrition, water or medication via the tube until confirmation the tube is in the correct place.



NOTES:

If it is not possible to obtain gastric aspirate to confirm the pH value, you could try the following:

- Lie your child on their left side, wait for 15 minutes, then try again to obtain gastric aspirate
- Give mouth care to your child to stimulate gastric secretion, then try again to obtain gastric aspirate
- If it is still not possible to obtain any gastric aspirate do not use the tube and contact a healthcare professional for further instructions

The Nutricia Homeward Nursing Service advice literature is provided as guidance following Nutricia Homeward Nursing Service training, and should not be used as a substitute for medical advice. Always contact your healthcare professional prior to making any changes to your tube feeding regimen.

Please contact the Nutricia Homeward Nurse or managing healthcare professional for the most up to date version every year. Provided on behalf of Nutricia Ltd at the request of the NHS managing healthcare professional.

IMPORTANT: Always seek urgent medical advice if there are any signs of abdominal pain, chest pain or breathing difficulties.

Nutricia Ltd
White Horse Business Park, Trowbridge
Wiltshire BA14 0XQ
nutriciafocare.com
SCC3649-03/19



HOMEWARD
NURSING SERVICE