

FLOCARE[®] INFINITY[™] II

INSTRUCTIONS FOR USE

PRE-PROGRAMMING SET-UP (FOR PACK OR OPTRI BOTTLE)



1 Wash hands thoroughly, check feed and expiry date



2 Attach FloCare Infinity Pack giving set to prescribed feed (or water)



3 Screw tightly



4 Open door and insert giving set



5 Close door (click)



6 Press and hold ON/OFF until beeps



7 Wait for VOLUME screen to be displayed (after calibration symbols)





8 Press CLR (short press)



9 Rate screen will be displayed, use +/- keys to programme rate




Please turn over for programming options

9A Programming Option 1 – Continuous feeding

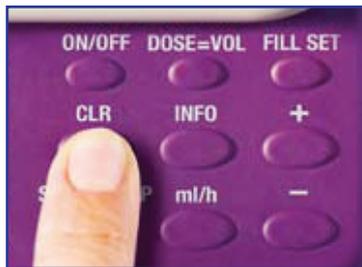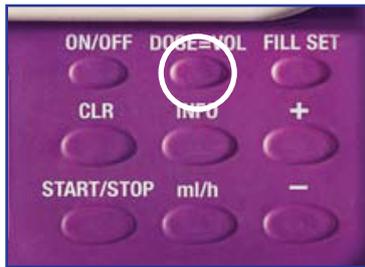

Press DOSE/VOLUME, then CLR, screen will show CONT


9B OR Programming Option 2 – Dose feeding

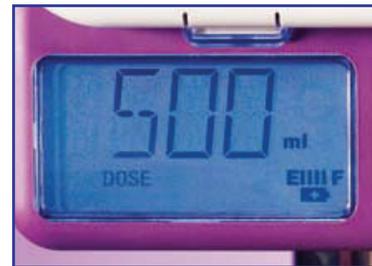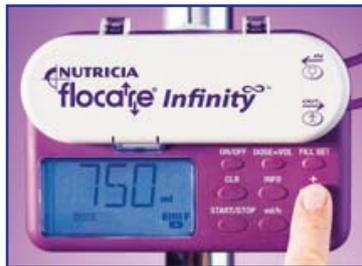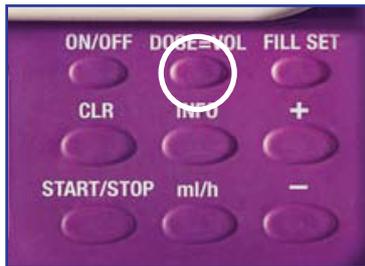

Press DOSE=VOL, use +/- keys to programme dose



Press and hold FILL SET until beeps;
to stop fill cycle press FILL SET again.
DO NOT OVERFILL

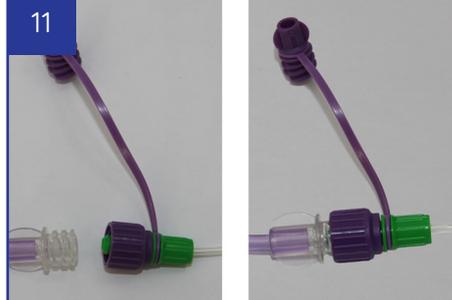

Connect giving set
to feeding tube



Press START/STOP
(3 beeps)



The RUN symbol will rotate to
indicate that the pump is working

For more information call 0800 093 3672
or for online training visit nutriciaflocare.com

Information contained here relates to programming the pump only. For clinical queries contact your healthcare professional. This information is intended for patients or carers of those who have been prescribed a tube feed by a healthcare professional.