

FLOCARE[®] INFINITY[™]

PICTORIAL GUIDE

PRE-PROGRAMMING SET-UP (FOR PACK OR OPTRI BOTTLE)



1 Wash hands thoroughly, check feed and expiry date



2 Attach Flocare Infinity Pack giving set to prescribed feed (or water)



3 Screw tightly



4 Open door and insert giving set



5 Close door (click)



6 Press and hold ON/OFF until beeps



7 Wait for VOLUME screen to be displayed (after calibration symbols)





8 Press CLR (short press)



9 Rate screen will be displayed, use +/- keys to programme rate



Please turn over for programming options

9A Programming Option 1 – Continuous feeding

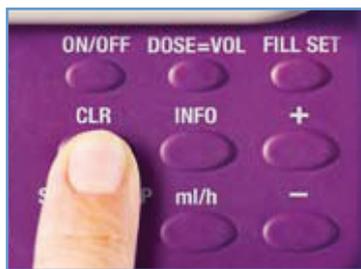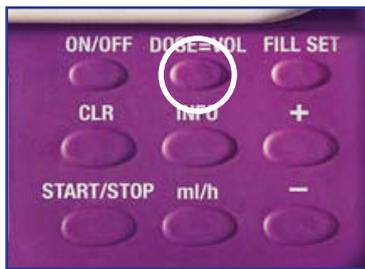


Press DOSE/VOLUME, then CLR, screen will show CONT

9B OR Programming Option 2 – Dose feeding

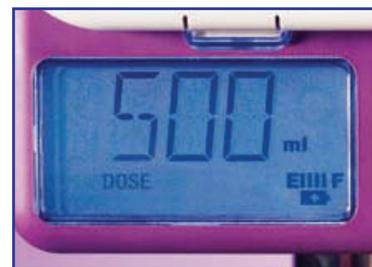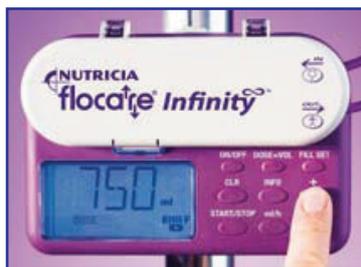

Press DOSE=VOL, use +/- keys to programme dose

10

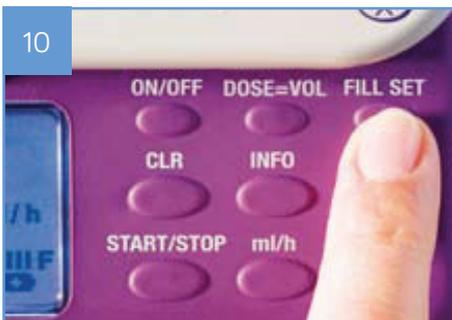


Press and hold FILL SET until beeps

11



Connect giving set to feeding tube

12

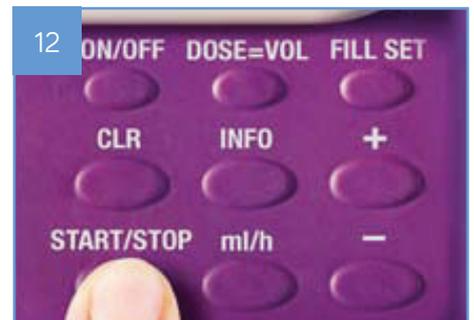

Press START/STOP (3 beeps)

13



The RUN symbol will rotate to indicate that the pump is working

For more information or online training visit
www.nutriciaflocare.com

Information contained here relates to programming the pump only. For clinical queries contact your healthcare professional.